

Seating System Armrests IPB, International

This Illustrated Parts Breakdown (IPB) Catalog has been compiled from the latest product information available at the time of publication. Quantum Rehab® / Pride Mobility Products® Corporation reserves the right to make changes and updates to this manual and to the product(s) it represents. Any changes, updates, or improvements to the product(s) may result in slight differences between this manual and the product that has been purchased.

An IPB is a graphically portrayed parts listing of product components which have been deemed to have serviceable, maintenance significance. When applicable, IPBs utilize “exploded” illustrations that detail serviceable components and a corresponding itemized parts list of reference numbers, part numbers, descriptions, quantities, and any applicable pricing. Most product components are incorporated into assemblies and may not be available for replacement.

Serviceable components will be noted by a callout in the illustration that will correspond to a alphanumeric reference number in the parts list. Items may also be listed in the parts list by parent assemblies. In most cases when a parent assembly is listed, there will be child parts or sub-assemblies listed under that parent.

For example: The parent assembly reference number is A1 — A5. That part contains all of the items listed as and contained within A1, A2, A3, A4, & A5. As long as these subcomponents have a callout associated with them, they can be selected for replacement or quote.

Updates to the contents of an IPB will be made on a regular basis based upon applicable product changes and enhancements. This PDF catalog has been export from the Provider based service site. The most up to date parts look up tool and ordering application is located at: prideprovider.com. Please utilize this system to enhance your user experience. It is built with highly interactive materials that are set with relational properties that will facilitate easier part look-up, more accurate parts selection, notification of onhand part availability, and quick links to related documents and products.

We welcome all questions and comments regarding the Quantum Rehab® / Pride Mobility Products® Corporation IPB systems.

Any pricing that is included in Quantum Rehab® /Pride Mobility Products® Corporation IPBs is subject to change without prior notice.

Thank you for choosing products from Quantum Rehab® and Pride Mobility Products® Corporation.



1.4. 2-Post Flip-up Armrests & Armpads

2-Post Flip-up Armrest, Desk Length, TB3/TB2/Synergy/Flex 1

2-Post Flip-up Armrest, Full Length, TB3/TB2/Synergy/Flex 3

2 Post Flip Up Armrest, Desk Length w/Waterfall Spacer, TB3/TB2/Synergy/Flex 5


2 Post Flip Up Armrest, Full Length w/Waterfall Spacer, TB3/TB2/Synergy/Flex 7

Appendix A:Pages Index 9

Appendix B:Parts Index 10

1.4. 2-Post Flip-up Armrests & Armpads





Only Serviceable Components Contain Callouts

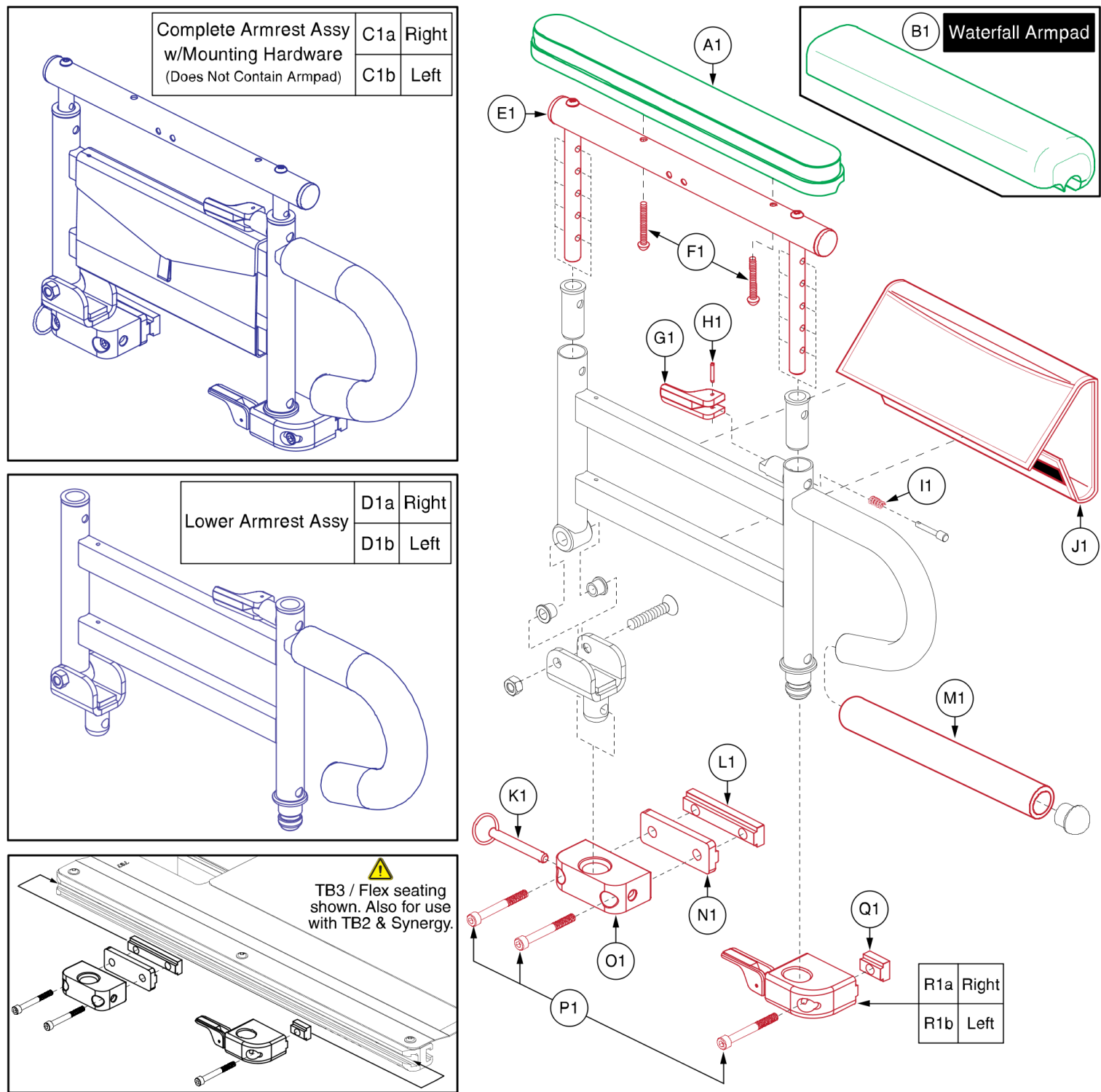

Item	Part No.	Name	Qty
A1	SETLRAM1115	ARMREST,PAD,SHORT,QUANTUM,V2	1
B1	SETLRAM1164	ARMPAD,ASSY,WATERFALL,BLACK,9", DESK LENGTH	1
C1a	FRMASMB7061	ARMREST,TWO POST FLIP-UP,DESK LENGTH,RIGHT	1
C1b	FRMASMB7060	ARMREST,TWO POST FLIP-UP,DESK LENGTH,LEFT	1
D1a	FRMASMB6793	ARMREST,TWO POST FLIP-UP,LOWER,RIGHT	1
D1b	FRMASMB6792	ARMREST,TWO POST FLIP-UP,LOWER,LEFT	1
E1	FRMASMB6967	ARMREST,TWO POST FLIP-UP,DESK LENGTH,UPPER	1
F1	SCRASBH1053	SCREW,BHSCS,BLACK ZINC,10-32 X 1.25"	2
G1	DWR1335B013	CAM LATCH,TWO POST FLIP UP,HEIGHT ADJUSTABLE ARMREST	1
H1	PINSPRG1018	SPRING PIN,0.0937" DIA X 0.625" L	1
I1		SPRING,.50"FL X .24"OD X .029" WIRE DIA	1
J1	DWR1010C003	POUCH,NYLON,TWO POST FLIP-UP ARMREST	1
K1	PINCOTD1034	COTTERLESS DETENT PIN,.3125"DIA X 2.50"LENGTH,W/RING	1
L1	DWR1360B015	T-NUT,BLACK,5/16-18 X 3", W/2",HOLE SPACING,B-1360-015	1
M1	PROHDL1062	HAND GRIP,BLACK,FOAM,.75"ID X 10.00"L X .25" THICKNESS	1
N1	DWR1335B019	SPACER,REAR,TWO POST FLIP-UP ARMREST	1
O1	DWR1335B012	ARMREST,BLOCK,REAR,TWO POST FLIP-UP ARMREST	1
P1	SCRSHCS1099	SCREW,SHCS,BZ,5/16-18 X 1.25",LOW HEAD,GR 8,W/PATCH	3
Q1	DWR1360A004	T-NUT,5/16-18 X 0.875",A-1360-004	1
R1a	FRMASMB6375	ARMREST,LATCH BLOCK,TWO POST FLIP-UP,FRONT,RIGHT	1
R1b	FRMASMB6374	ARMREST,LATCH BLOCK,TWO POST FLIP-UP,FRONT,LEFT	1

1.4. 2-Post Flip-up Armrests & Armpads

Only Serviceable Components Contain Callouts

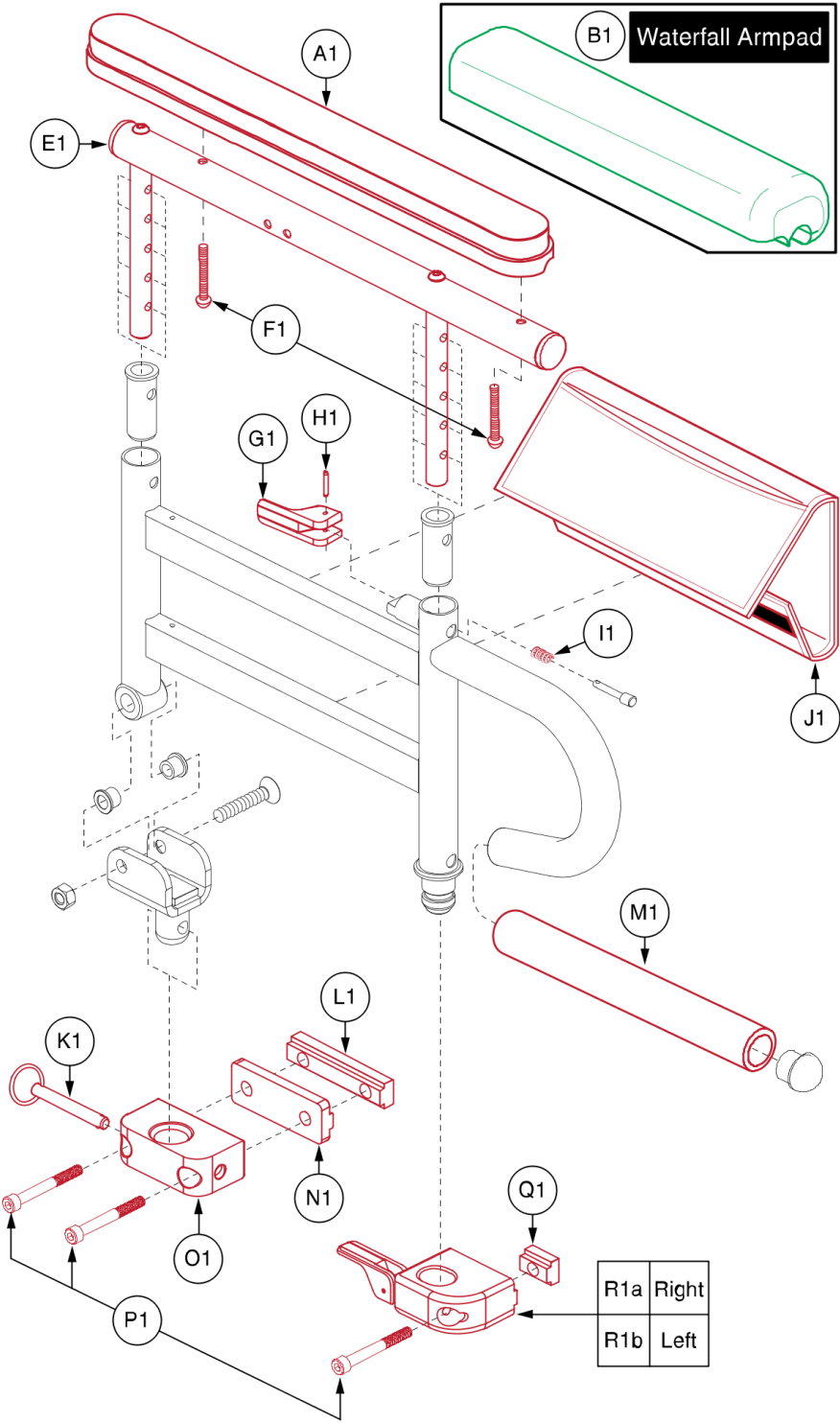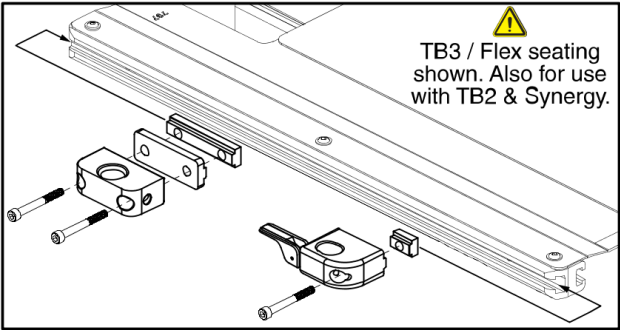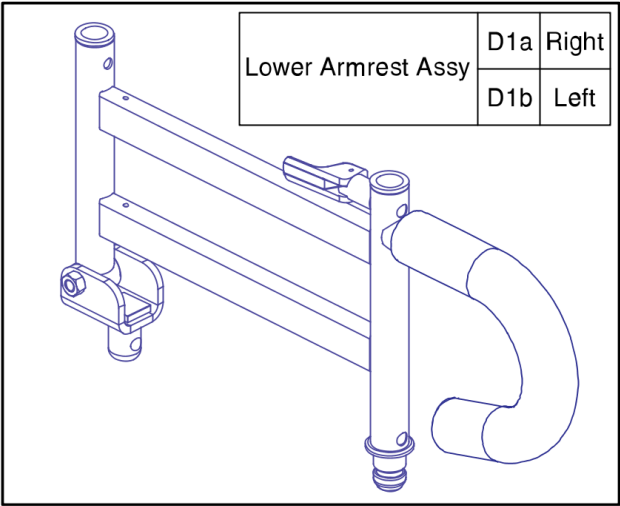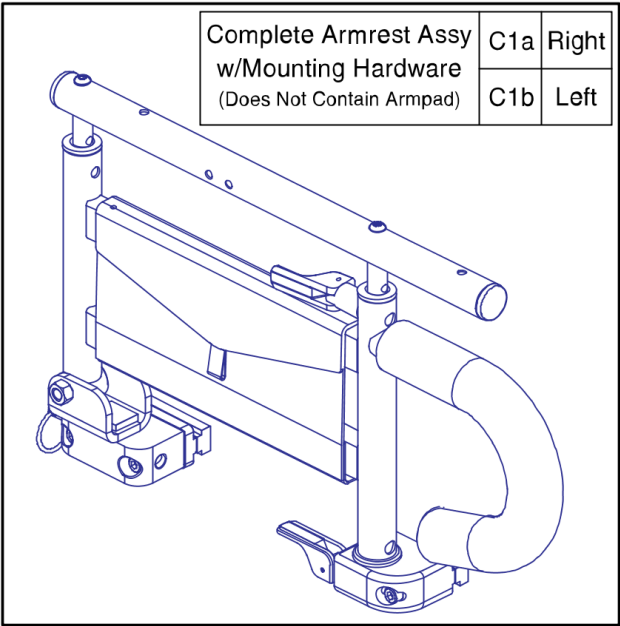




Item	Part No.	Name	Qty
A1	SETLRAM1116	ARMREST,PAD,LONG,QUANTUM,V2	1
B1	SETLRAM1165	ARMPAD,ASSY,WATERFALL,BLACK,14", FULL LENGTH	1
C1a	FRMASMB7063	ARMREST,TWO POST FLIP-UP,FULL LENGTH,RIGHT	1
C1b	FRMASMB7062	ARMREST,TWO POST FLIP-UP,FULL LENGTH,LEFT	1
D1a	FRMASMB6793	ARMREST,TWO POST FLIP-UP,LOWER,RIGHT	1
D1b	FRMASMB6792	ARMREST,TWO POST FLIP-UP,LOWER,LEFT	1
E1	FRMASMB6968	ARMREST,TWO POST FLIP-UP,FULL LENGTH,UPPER	1
F1	SCRASBH1053	SCREW,BHSCS,BLACK ZINC,10-32 X 1.25"	2
G1	DWR1335B013	CAM LATCH,TWO POST FLIP UP,HEIGHT ADJUSTABLE ARMREST	1
H1	PINSPRG1018	SPRING PIN,0.0937" DIA X 0.625" L	1
I1		SPRING,.50"FL X .24"OD X .029" WIRE DIA	1
J1	DWR1010C003	POUCH,NYLON,TWO POST FLIP-UP ARMREST	1
K1	PINCOTD1034	COTTERLESS DETENT PIN,.3125"DIA X 2.50"LENGTH,W/RING	1
L1	DWR1360B015	T-NUT,BLACK,5/16-18 X 3", W/2",HOLE SPACING,B-1360-015	1
M1	PROHDL1062	HAND GRIP,BLACK,FOAM,.75"ID X 10.00"L X .25" THICKNESS	1
N1	DWR1335B019	SPACER,REAR,TWO POST FLIP-UP ARMREST	1
O1	DWR1335B012	ARMREST,BLOCK,REAR,TWO POST FLIP-UP ARMREST	1
P1	SCRSHCS1099	SCREW,SHCS,BZ,5/16-18 X 1.25",LOW HEAD,GR 8,W/PATCH	3
Q1	DWR1360A004	T-NUT,5/16-18 X 0.875",A-1360-004	1
R1a	FRMASMB6375	ARMREST,LATCH BLOCK,TWO POST FLIP-UP,FRONT,RIGHT	1
R1b	FRMASMB6374	ARMREST,LATCH BLOCK,TWO POST FLIP-UP,FRONT,LEFT	1

1.4. 2-Post Flip-up Armrests & Armpads


Only Serviceable Components Contain Callouts

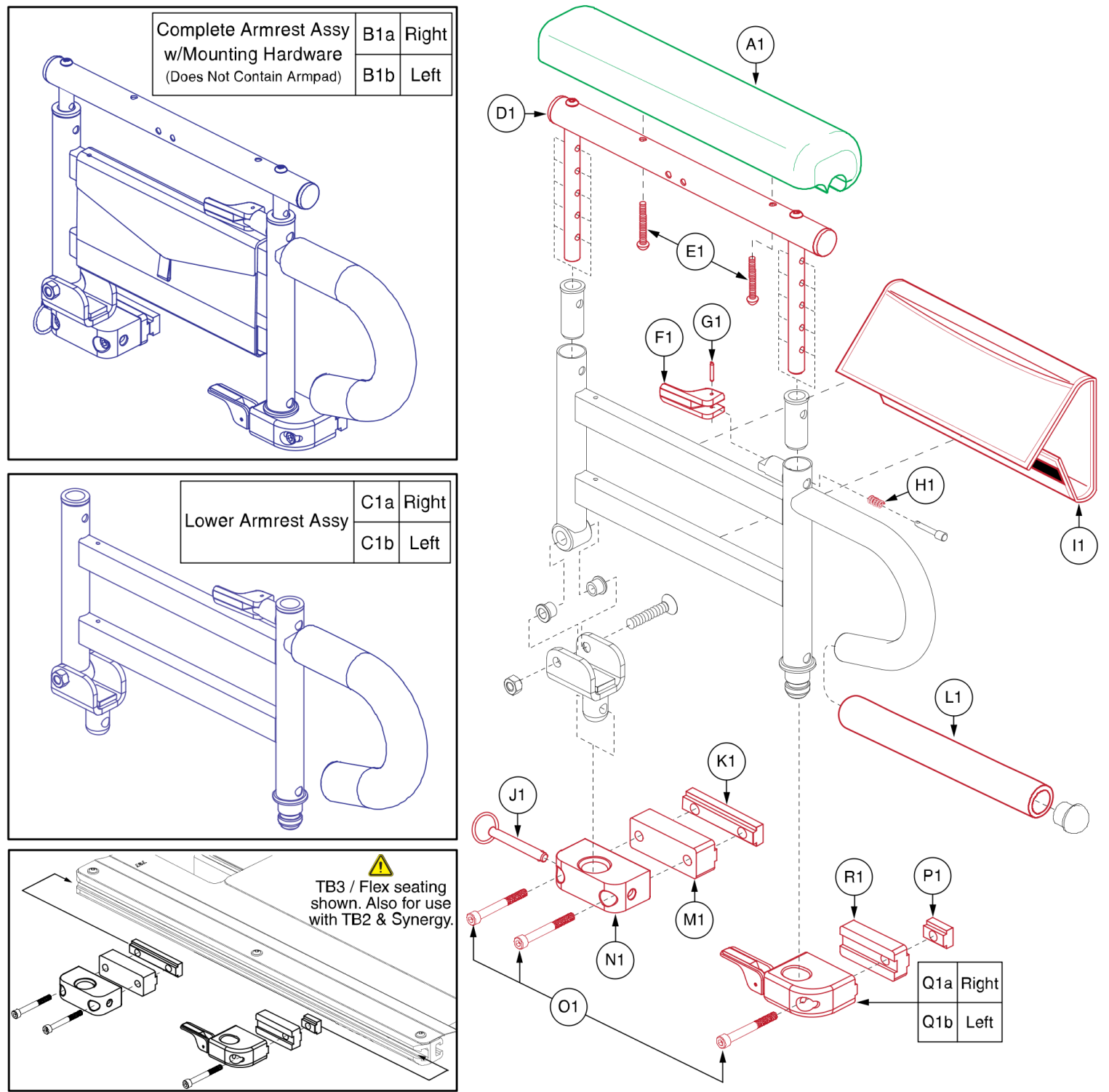

Item	Part No.	Name	Qty
A1	SETLRAM1164	ARMPAD,ASSY,WATERFALL,BLACK,9", DESK LENGTH	1
B1a	FRMASMB15967	ARMREST,TWO POST FLIP-UP,DESK LENGTH,W/WATERFALL SPACERS,RIGHT	1
B1b	FRMASMB15966	ARMREST,TWO POST FLIP-UP,DESK LENGTH,W/WATERFALL SPACERS,LEFT	1
C1a	FRMASMB6793	ARMREST,TWO POST FLIP-UP,LOWER,RIGHT	1
C1b	FRMASMB6792	ARMREST,TWO POST FLIP-UP,LOWER,LEFT	1
D1	FRMASMB6967	ARMREST,TWO POST FLIP-UP,DESK LENGTH,UPPER	1
E1	SCRASBH1053	SCREW,BHSCS,BLACK ZINC,10-32 X 1.25"	2
F1	DWR1335B013	CAM LATCH,TWO POST FLIP UP,HEIGHT ADJUSTABLE ARMREST	1
G1	PINSPRG1018	SPRING PIN,0.0937" DIA X 0.625" L	1
H1		SPRING,.50"FL X .24"OD X .029" WIRE DIA	1
I1	DWR1010C003	POUCH,NYLON,TWO POST FLIP-UP ARMREST	1
J1	PINCOTD1034	COTTERLESS DETENT PIN,.3125"DIA X 2.50"LENGTH,W/RING	1
K1	DWR1360B015	T-NUT,BLACK,5/16-18 X 3", W/2",HOLE SPACING,B-1360-015	1
L1	PROHDL1062	HAND GRIP,BLACK,FOAM,.75"ID X 10.00"L X .25" THICKNESS	1
M1	DWR1500B069	SPACER,TWO-POST,REAR,B-1500-069	1
N1	DWR1335B012	ARMREST,BLOCK,REAR,TWO POST FLIP-UP ARMREST	1
O1	SCRSHCS1155	SCREW,SHCS,BZ,5/16-18 X 1.75",LOW HEAD,GR 8,W/PATCH	3
P1	DWR1360A004	T-NUT,5/16-18 X 0.875",A-1360-004	1
Q1a	FRMASMB6375	ARMREST,LATCH BLOCK,TWO POST FLIP-UP,FRONT,RIGHT	1
Q1b	FRMASMB6374	ARMREST,LATCH BLOCK,TWO POST FLIP-UP,FRONT,LEFT	1
R1	DWR1500B070	SPACER,TWO-POST,FRONT,B-1500-070	1

1.4. 2-Post Flip-up Armrests & Armpads

Only Serviceable Components Contain Callouts

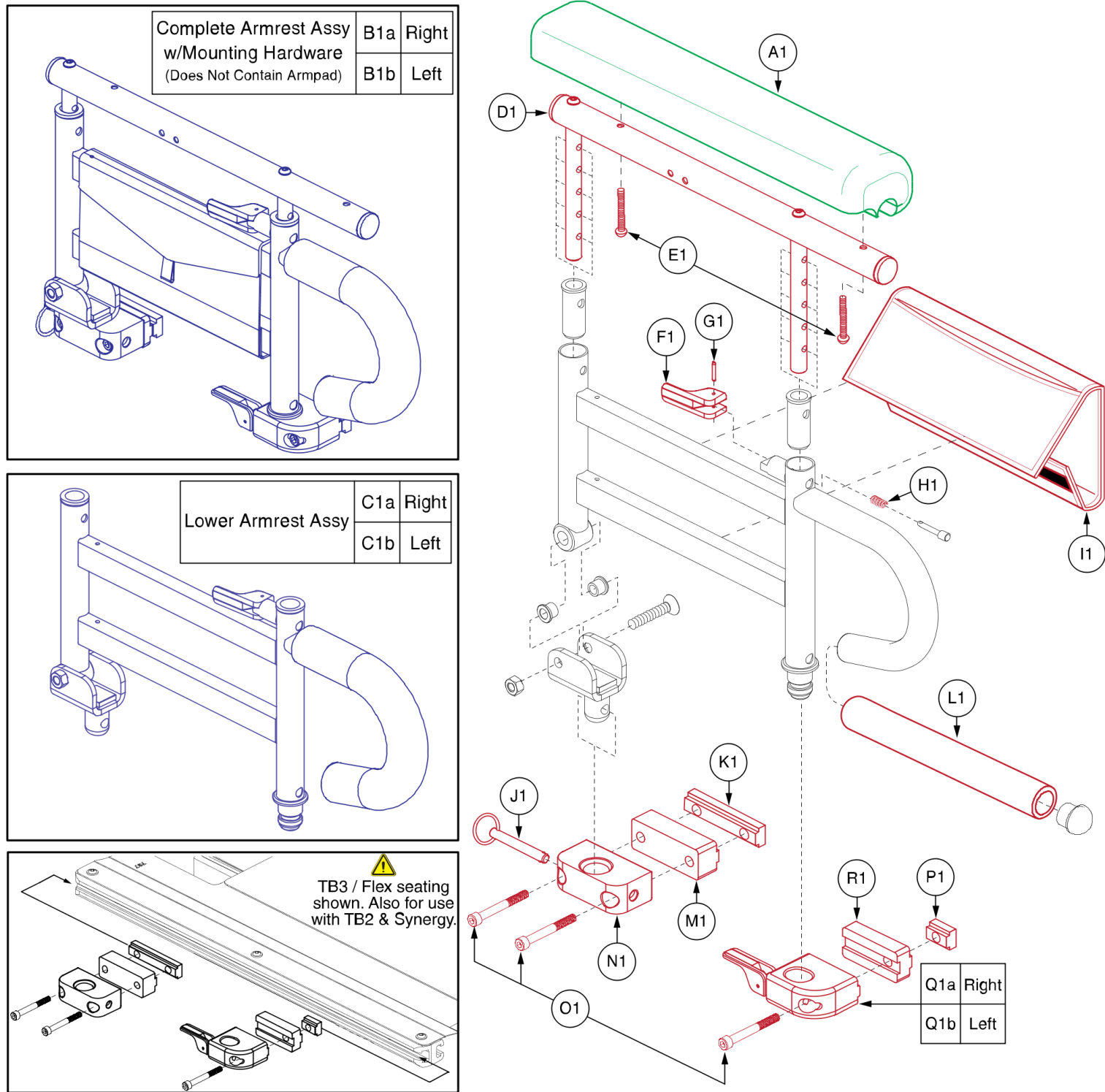

Item	Part No.	Name	Qty
A1	SETLRAM1165	ARMPAD,ASSY,WATERFALL,BLACK,14", FULL LENGTH	1
B1a	FRMASMB15969	ARMREST,TWO POST FLIP-UP,FULL LENGTH,W/WATERFALL SPACERS,RIGHT	1
B1b	FRMASMB15968	ARMREST,TWO POST FLIP-UP,FULL LENGTH,W/WATERFALL SPACERS,LEFT	1
C1a	FRMASMB6793	ARMREST,TWO POST FLIP-UP,LOWER,RIGHT	1
C1b	FRMASMB6792	ARMREST,TWO POST FLIP-UP,LOWER,LEFT	1
D1	FRMASMB6968	ARMREST,TWO POST FLIP-UP,FULL LENGTH,UPPER	1
E1	SCRASBH1053	SCREW,BHSCS,BLACK ZINC,10-32 X 1.25"	2
F1	DWR1335B013	CAM LATCH,TWO POST FLIP UP,HEIGHT ADJUSTABLE ARMREST	1
G1	PINSPRG1018	SPRING PIN,0.0937" DIA X 0.625" L	1
H1		SPRING,.50"FL X .24"OD X .029" WIRE DIA	1
I1	DWR1010C003	POUCH,NYLON,TWO POST FLIP-UP ARMREST	1
J1	PINCOTD1034	COTTERLESS DETENT PIN,.3125"DIA X 2.50"LENGTH,W/RING	1
K1	DWR1360B015	T-NUT,BLACK,5/16-18 X 3", W/2",HOLE SPACING,B-1360-015	1
L1	PROHDL1062	HAND GRIP,BLACK,FOAM,.75"ID X 10.00"L X .25" THICKNESS	1
M1	DWR1500B069	SPACER,TWO-POST,REAR,B-1500-069	1
N1	DWR1335B012	ARMREST,BLOCK,REAR,TWO POST FLIP-UP ARMREST	1
O1	SCRSHCS1155	SCREW,SHCS,BZ,5/16-18 X 1.75",LOW HEAD,GR 8,W/PATCH	3
P1	DWR1360A004	T-NUT,5/16-18 X 0.875",A-1360-004	1
Q1a	FRMASMB6375	ARMREST,LATCH BLOCK,TWO POST FLIP-UP,FRONT,RIGHT	1
Q1b	FRMASMB6374	ARMREST,LATCH BLOCK,TWO POST FLIP-UP,FRONT,LEFT	1
R1	DWR1500B070	SPACER,TWO-POST,FRONT,B-1500-070	1

Appendix A:Pages Index

2 Post Flip Up Armrest, Desk Length w/Waterfall Spacer, TB3/TB2/Synergy/Flex	5
2 Post Flip Up Armrest, Full Length w/Waterfall Spacer, TB3/TB2/Synergy/Flex	7
2-Post Flip-up Armrest, Desk Length, TB3/TB2/Synergy/Flex	1
2-Post Flip-up Armrest, Full Length, TB3/TB2/Synergy/Flex	3

Appendix B:Parts Index

DWR1010C003	2, 4, 6, 8
DWR1335B012	2, 4, 6, 8
DWR1335B013	2, 4, 6, 8
DWR1335B019	2, 4
DWR1360A004	2, 4, 6, 8
DWR1360B015	2, 4, 6, 8
DWR1500B069	6, 8
DWR1500B070	6, 8
FRMASMB15966	6
FRMASMB15967	6
FRMASMB15968	8
FRMASMB15969	8
FRMASMB6374	2, 4, 6, 8
FRMASMB6375	2, 4, 6, 8
FRMASMB6792	2, 4, 6, 8
FRMASMB6793	2, 4, 6, 8
FRMASMB6967	2, 6
FRMASMB6968	4, 8
FRMASMB7060	2
FRMASMB7061	2
FRMASMB7062	4
FRMASMB7063	4
PINCOTD1034	2, 4, 6, 8
PINSPRG1018	2, 4, 6, 8
PROHDL1062	2, 4, 6, 8
SCRASBH1053	2, 4, 6, 8
SCRSHCS1099	2, 4
SCRSHCS1155	6, 8
SETLRAM1115	2
SETLRAM1116	4
SETLRAM1164	2, 6
SETLRAM1165	4, 8
SPGRAW1065	2, 4, 6, 8

Illustrated Parts Breakdown

USA - Quantum Rehab

401 York Avenue
Duryea, Pa 18642
www.quantumrehab.com

USA - Pride Mobility Products Corp.

182 Susquehanna Avenue
Exeter, Pa 18643
www.pridemobility.com

Canada

5096 South Service Road
Beamsville, Ontario L0R 1B3
www.pridemobility.ca
www.quantumrehab.com

Australia

20-24 Apollo Drive
Hallam, Victoria 3803
www.pridemobility.com.au
www.quantumrehab.com.au

New Zealand

38 Lansford Crescent
Avondale, Auckland 0600
www.pridemobility.co.nz

UK

32 Wedgwood Road
Bicester, Oxfordshire OX26 4UL
www.pride-mobility.co.uk
www.quantumrehab.co.uk

Netherlands

(Authorised EU Representative)

De Zwaan 3
1601 MS Enkhuizen
www.pridemobility.eu
www.quantumrehab.eu

Italy

Via del Progresso, ang. Via del Lavoro
Loc. Prato della Corte
00065 Fiano Romano (RM)
www.pride-italia.it
www.quantumrehab-italia.it

France

26 Rue Monseigneur Ancel
69800 Saint-Priest
www.pridemobility.fr
www.quantumrehab.fr

Spain

Calle Las Minas Número 67
Polígono Industrial Urtinsa II, de Alcorcón
28923 Madrid

Germany

Hövelrieger Str. 28
33161 Hövelhof
www.pridemobility.de

China

Room 508, Building #4
TianNa Business Zone
No. 500 Jianyun Road
Pudong New Area
Shanghai 201318

