



ALLIED
MEDICAL

USER MANUAL

Contoured Bed Pan



SLBA7242

User Instructions

INTRODUCTION

The Bed Pan is a plastic container used to collect urine or feces. It is useful for people who need to use the bathroom without leaving their bed. The built-in handles on the sides are easy for transportation.

SPECIFICATION

Size	Weight	Material	Max. Load Weight
390x265x45mm	405g	PE	150kg

INSTRUCTIONS FOR USE

1. Lift the user into a seated position or gently lift up the user's hips.
2. Slide the bedpan under the person.
3. After use, slide the bedpan away and dispose the contents.
4. Clean the bedpan.

ATTENTION

- Heat safe up to 50°C
- Do not disinfect the Bed Pan with high temperature and pressure.
- The max. load weight is 150kgs.
- Periodically inspect for scrape, wear and damage.

MAINTENANCE

Clean with mild detergent and a brush.

Do not exceed 50°C